AED-Alert™ 2.0

Sistema di Gestione remota DAE

Connessi & Sicuri

mindray
healthcare within reach

Sommario

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Home Device Mgmt. Event Mgmt. Pos. Offset Permission Mgmt. System Mgmt.

Current Pos. : Device Mgmt. > Device Details

Device SN: AE7-97000072 Device Model: All Branch: All Customer Name: Enter Search Reset&Return

Basic Info Warning Fault Self-test report Device Log Rescue summary Rescue log Power on/off log

Basic Info

Device SN : AE7-97000072
Device Model : BeneHeart C2

[More Info](#)

Device Status

Normal

Battery Status

Remaining capacity 98%

Last Battery Replacement Time : 2019-08-16

Pads Status

21days

Pads expire on 2019-10-01

Pos. Status

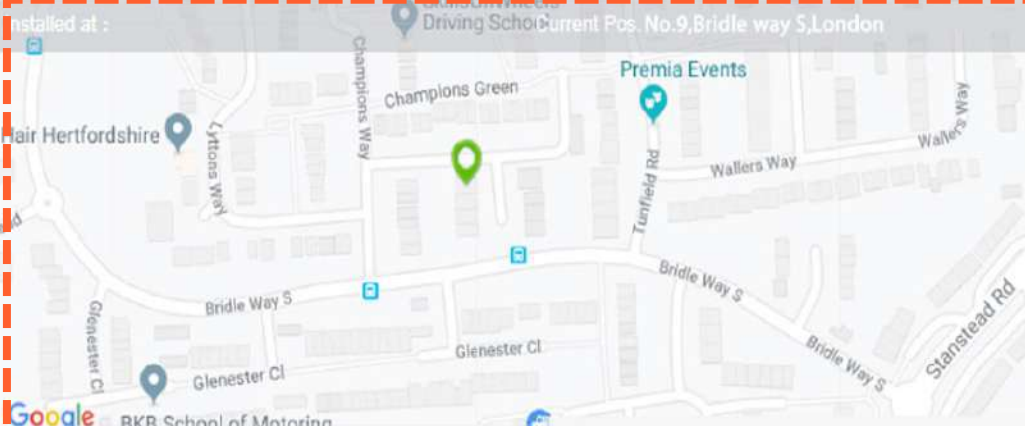
Normal

Network Status

Good Signal

Installed at : 

Driving School Current Pos. No.9,Bridle way S,London

Premia Events

Champions Green

Champions Way

Waller's Way

Tunfield Rd

Bridle Way S

Glenester Cl

BKB School of Motoring

Delete Image

mindray

AED

Upload Image

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Device SN : AE7-97000072
Device Model : BeneHeart C2

Device Status

Normal

Self-Test Result Details- (Device SN:LY-WDF-002)

Test Item	Test Result	Remarks
Control module	Pass	
ECG module	Pass	
Treatment module	Pass	
1J charge/discharge	Pass	
Battery	Pass	Battery SN:HY09191450347 Battery Validity(Year/Month):202404 Remaining Power:6054mAh
Pads	Pass	Validity:2024-09-12
Signal type:4G	Signal	RSRP:-84 dBm

Close

Delete Image Upload Image

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Mindray IoT - AED ALERT

https://aedalert.mindray.com

mindray 迈瑞

Home Device Mgmt. Event Mgmt. Pos. Offset Permission Mgmt. System Mgmt.

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Basic Info Warning Fault Self-test report Device Log Rescue summary Rescue log Power on/off log

Basic Info

Device SN : AE7-97000072
Device Model : BeneHeart C2
More Info

Device Status

Remaining capacity of battery

Battery Status

Remaining capacity 98%
Last Battery Replacement Time : 2019-08-16

Pads Status

21days
Pads expire on 2019-10-01

Pos. Status

Remaining time & expiry date of pads

Network Status

Good Signal

Interact Map

Driving School from Local Police Academy, London

Champions Green

Premia Events

Waller's Way

Tunfield Rd

Bride Way S

Glenester Cl

Stamstead Rd

Google BKB School of Motoring

Delete Image

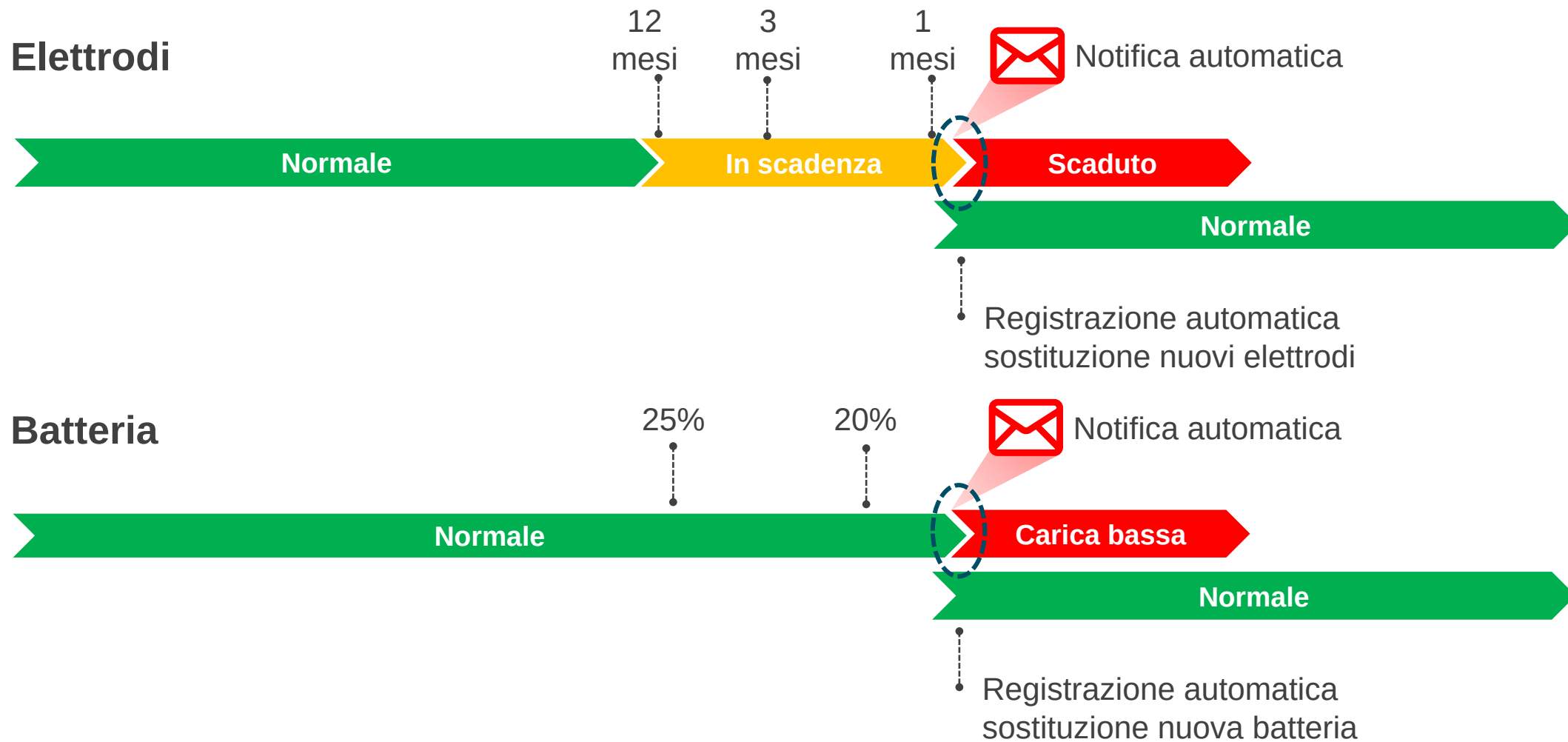
mindray

AED

Upload Image

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mindray 迈瑞

Home Device Mgmt. Event Mgmt. Pos. Offset Permission Mgmt. System Mgmt.

Current Pos. Device Mgmt. Device Details

Device SN: AE7-97000072 Device Model: BeneHeart C2

Basic Info Warning Fault Self-test report Device Log Rescue summary Rescue log Power on/off log

Basic Info

Device SN : AE7-97000072
Device Model : BeneHeart C2
[More Info](#)

Device Status

Normal

Battery Status

Remaining capacity 98%
Last Battery Replacement Time : 2019-08-16

Pads Status

21days
Pads expire on 2019-10-01

Pos. Status

Normal

Network Status

Good Signal

Installed at :
Driving School Current Pos. No.9,Bridle way S,London

Champions Green
Champions Way
Premia Events
Tunfield Rd
Wallera Way
Bridle Way S
Glenester Cl
Starstead Rd

Google BKB School of Motoring

Delete Image

Upload Image

AED

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Mindray IoT - AED ALERT

https://aedalert.mindray.com

mindray.com

Home Device Mgmt. Event Mgmt. Pos. Offset Permission Mgmt. System Mgmt.

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Model All Branch All Customer Name Enter Search Reset&Return

Basic Info Warning Fault Self-test report Device Log Rescue summary Rescue log Power on/off log

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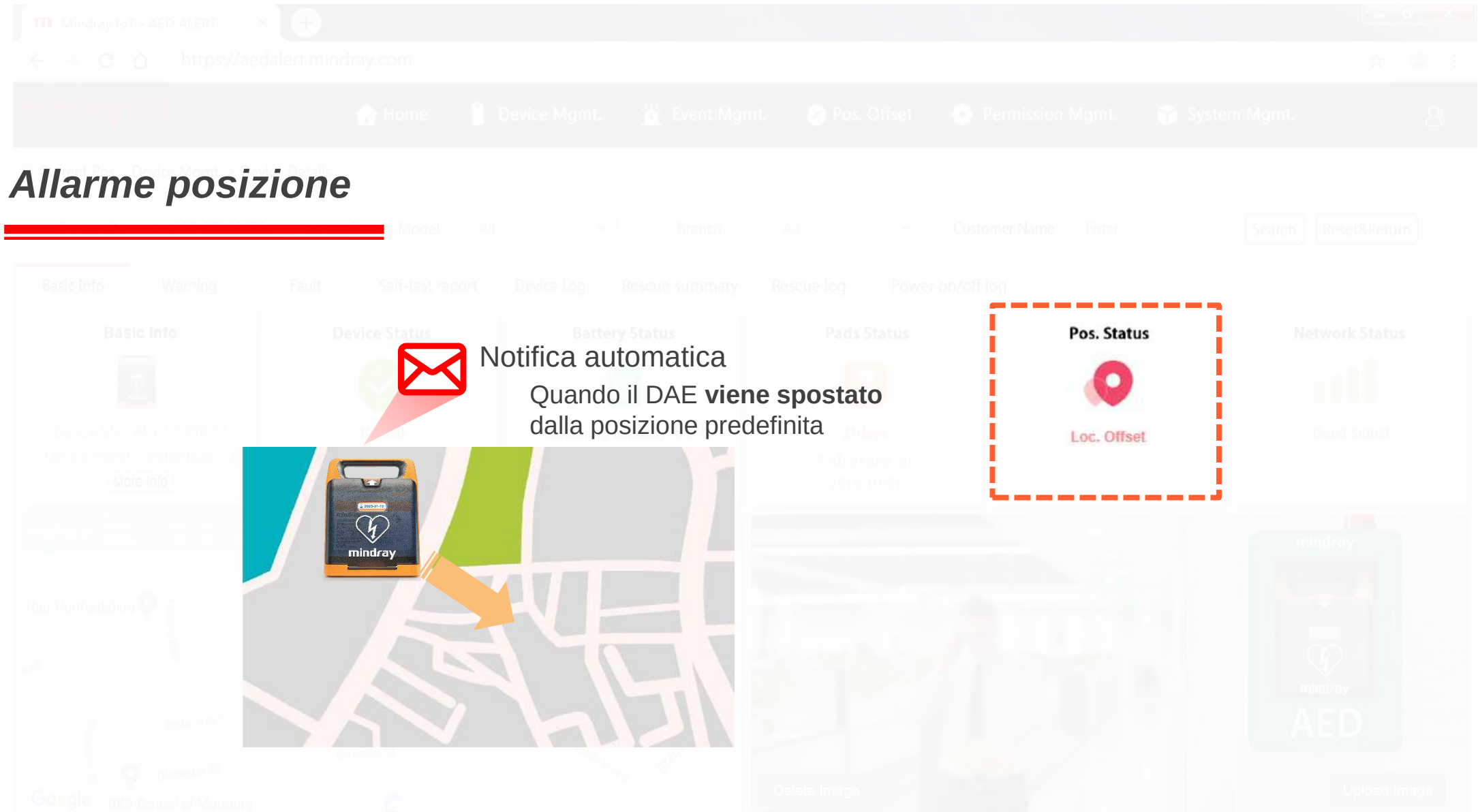
mindray 迈瑞

Home Device Mgmt. Event Mgmt. Pos. Offset Permission Mgmt. System Mgmt.

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
Device Model: All Branch: All Customer Name: Enter Search Reset&Return

Basic Info Warning Fault Self-test report Device Log Rescue summary Rescue log Power on/off log

Basic Info	Device Status	Battery Status	Pads Status	Pos. Status	Network Status
 Device ID: 2019-08-10 More Info	 Normal	 Remaining Capacity: 2019-08-10	 21days Pads expired: 2019-10-01	 Normal	 Good Signal

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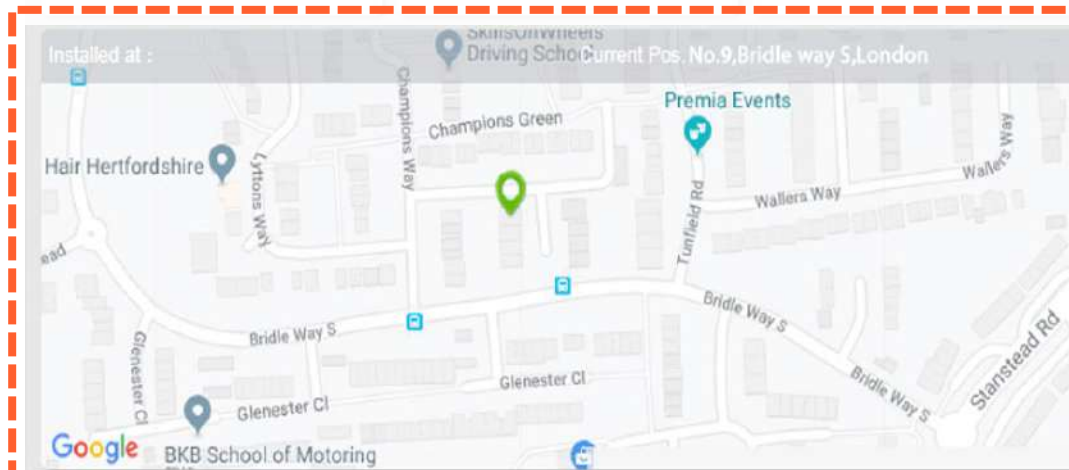Gestore del DAE /
Manutentore

Rescue Summary Rescue Log Power-On/Off Log

~ yy-mm-dd Search Reset & Return

n	Rescue Time(s)	CPR Duration(s)	Shocks	Received Time(Berlin T...
00.08	116	90	1	2019-08-28 11:56:20
00.08	189	82	1	2019-08-28 11:15:19

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Device SN: Zoll-57254453	Device Model: AED plus
Branch:	Asset SN: ZC-172574568
Client Name (CRM):	• Manually typing in device & readiness information
Client Name: William Huang	Device Admin:
Inst. Time: 2019-08-25	Self-test Period (Day):
Self-test sending cycle (Day):	Last Upload Time: Auto-alert -when pads expired
Device State: Fault Normal Fault ✓ ✕	Pads Service Life: 2021-08-11
Current Bound Agent: Fault	

Email di allarme

Notifica automatica

Dear user:

Your AED device failed self-test at 00:12:28 2019-08-30!

Device SN	Device Model	Installation loc.	Device State
1 ym-wdf-01	BeneHeart C2	Hai Si Lu, Nanshan Qu, Shenzhen Shi, Guangdong Sheng, China	Self-test failed,Fault Code:442;Detailed Failure Module:Main control module.

Access <https://aedalert.mindray.com> for details and conduct onsite inspection in time.

Tips:

- (1) View and manage device anomalies in time to avoid emergency rescue failure due to untimely anomaly management.
- (2) Remember the SN of your AED device to make sure you have follow-up service.
- (3) The AED device performs self-test periodically based on the setting. Once any device anomaly is found, it immediately notifies you by SMS message and email until you rectify the anomaly and the self-test succeeds.
- (4) Replacement parts, accessories, and consumables must be original or Mindray-approved.
- (5) In special cases, contact your dealer or Mindray CSC.
- (6) Follow the product manual for detailed operations. For more guidance on AED O&M, scan the QR code on the right to follow the Mindray CS video and watch the AED O&M video.

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